

Haptoglobin

CPT Code: 83010

Order Code: C2410

ABN Requirement: No

Synonyms: HPT; Hemoglobin-binding Protein; Hp

Specimen: Serum

Volume: 1.0 mL

Minimum Volume: 0.4 mL

Container: Gel-barrier tube (SST, Tiger Top)

Collection:

1. Collect and label sample according to standard protocols.
2. Gently invert tube 5 times immediately after draw. DO NOT SHAKE.
3. Allow blood to clot 30 minutes.
4. Centrifuge for 10 minutes.

Transport: Store serum at 2°C to 8°C after collection and ship the same day per packaging instructions provided with the Cleveland HeartLab shipping box.

Stability:

Ambient (15-25°C): 3 months

Refrigerated (2-8°C): 8 months

Frozen (-20°C): Not Acceptable

Deep Frozen (-70°C): Not Acceptable

Causes for Rejection: Samples that are grossly hemolyzed or lipemic; specimens other than serum; improper labeling; samples not stored properly; samples older than stability limits

Methodology: Turbidimetric Immunoassay (TUI)

Turn Around Time: 4 days

Reference Range:

Age	mg/dL
-----	-------

All Ages	31-238
----------	--------

Intended Use: Evaluation of intravascular hemolysis and acute or chronic inflammatory response.

The CPT codes provided are based on AMA guidelines and are for informational purposes only. CPT coding is the sole responsibility of the billing party. Please direct any questions regarding coding to the payer being billed.