

Protein, Urine, Random

CPT Code: 84156

Order Code: C2714

ABN Requirement: No

Specimen: Urine

Volume: 5.0 mL

Minimum Volume: 1.0 mL

Container: Urine (Yellow top, no preservative) tube

Collection:

Urine:

1. Collect and label urine sample according to standard protocols.
2. Transfer urine from a clean urine cup into a Yellow Top tube using the vacutainer transfer device included with the Yellow Top tube.
3. Cap tube tightly.
4. Label sample according to standard protocols.

Transport: Store urine at 2-8°C after collection and ship the same day per packaging instructions provided with the Cleveland HeartLab shipping box.

Stability:

Ambient (15-25°C): 1 day

Refrigerated (2-8°C): 7 days

Frozen (-20°C): 1 month

Deep Frozen (-70°C): 1 month

Causes for Rejection: Specimens other than random urine without preservatives; improper labeling; samples not stored properly; samples older than stability limits

Methodology: Turbidimetric Immunoassay (TUI)

Turn Around Time: 2-3 days

Reference Range:

Use: Detection of clinically significant proteinuria.

The CPT codes provided are based on AMA guidelines and are for informational purposes only. CPT coding is the sole responsibility of the billing party. Please direct any questions regarding coding to the payer being billed.